



Labor+ Technik

EBERHARD LEHMANN GmbH

Diagnostics and Labware
Consulting · Training · Customer Support

ORIGINAL

HAEMA – Quick-Stain **LT-SYS**[®] (Diff-Quick)

- **Easy handling – brilliant staining after only 1 minute**
- **Three-Step-Procedure Stain kit containing 3 ready-to-use solutions**
- **Staining results equivalent to Pappenheim method**
- **Provides consistent, reproducible staining quality**
- **Clear, differential results**
- **Rapid staining for haematology, cytology and histology**
- **High stability, excellent yield and colour intensity**
- **For human and veterinary labs**
- **Sperm morphology staining (Diff-Quick recommended by the WHO)**



With Haema-Quick-Stain **LT-SYS**[®] numerous specimen slides can be processed simultaneously, thus making it suitable for use in industrial Haematology laboratories, private practices and for emergency diagnosis. **Also useable in automatic staining systems.**

PROCEDURE

Prepare smear slide with blood or other specimen according to standard procedure:

1. Dip slide into Fixing Solution 5 times for 1 second, allowing excess reagent to drain.
2. Dip slide into Colour Reagent I, red 3 - 5 times for 1 second, allowing excess reagent to drain.
3. Dip slide into Colour Reagent II, blue 3 - 5 times for 1 second, allowing excess reagent to drain.
4. Rinse slide with distilled water and allow to dry.

**made
in
Germany**

PRODUCT DATA

LT 005	Haema-Quick Stain LT-SYS [®] , Set with Fixing Solution, Colour Reagent I and II, each 100 ml
LT 001	Haema-Quick Stain LT-SYS [®] , Set with Fixing Solution, Colour Reagent I and II, each 500 ml
LT 008/S	Haema-Quick Stain LT-SYS [®] , 1000 ml Fixing Solution
LT 003	Haema-Quick Stain LT-SYS [®] , 2,5 l Colour Reagent I, red
LT 004	Haema-Quick Stain LT-SYS [®] , 2,5 l Colour Reagent II, blue
1210	Dye-bath Set includes: 3 Glass Troughs with cover, 1 Stainless Steel Rack, 1 Glass Trough approx. 27,5 x 15 x 6 cm.
2407/1	Microscope Slides, 76 x 26 mm each, frosted ends, pack of 50 pcs.
LT-FLIO	Immersion Oil, 100 ml

Corresponding to the In-vitro-Diagnostic guideline 98/79/EC our products are CE registered. Our Quality Management System is certified by the TÜV according to EN ISO 13485:2016.

If you are interested in our products, please contact us:

LABOR + TECHNIK
Eberhard Lehmann GmbH

D-14167 Berlin
Tel.: 030 / 80 90 26 63
Fax: 030 / 80 90 26 65

LABOR + TECHNIK
Eberhard Lehmann GmbH

A-5020 Salzburg
Tel.: 06 62 / 43 94 40
Fax: 06 62 / 43 00 51

LT-SYS[®]

www.lt-sys.de
E-Mail: info@lt-sys.de

MORE THAN 45 YEARS: HIGH QUALITY CLINICAL REAGENTS AT LOW PRICES!